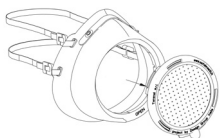
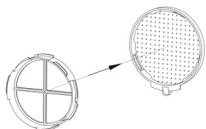
Istruzioni / Instructions

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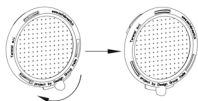
Remove the rings from the face mask.

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
Place the filter on the external ring of the SAFE BREATH mask (previously sanitized).

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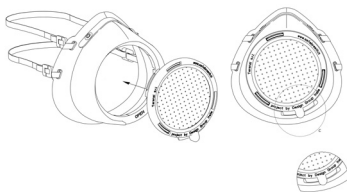Place the inner tightening ring on the filter and turn external ring clockwise (bayonet lock).

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Insert in the mask the two rings thus joined together.

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SAFE BREATH is a TGroup Terenzi and Design Group Italia project

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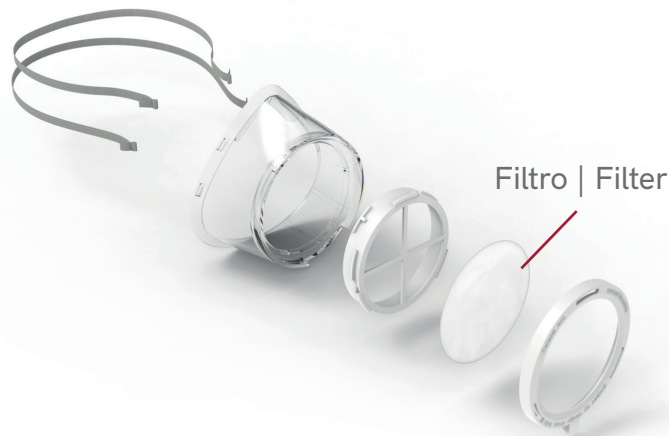
Refill Filters Safe Breath

Contents: refill filters

The filter is intended for use only with the CE certified mask SAFE BREATH Medical Device. It is not compatible with other products. Any use other than that indicated is to be considered improper.

Filter Features:

- filter compliant with EN 14683 technical standard classified as Type IIR;
- BFE > 98%;
- Replace the filter after each day of use (12h/approx).

How to Apply/Replace the Filter:

- Wash and sanitize your hands thoroughly;
- Remove the filter from its packaging;
- Remove the rings from the face mask;
- Place the filter on the external ring of the SAFE BREATH mask (previously sanitized);
- Place the inner tightening ring on the filter and turn external ring clockwise (bayonet lock);
- Insert in the mask the two rings thus joined together;
- Discard the filter after use it, if possible in the appropriate collector for the disposal of sanitary material and in any case according to the indications of your municipality. Do not disperse in the environment after use;
- Wash your hands thoroughly.

For further information, carefully consult the instructions for use supplied with the SAFE BREATH medical device.