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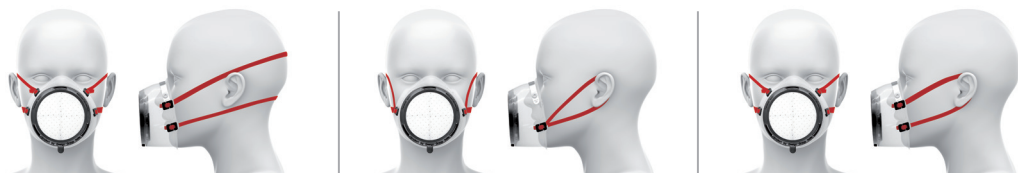
SAFE BREATH is a TGroup Terenzi and Design Group Italia project

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Protection mask with filter. Indicated to limit the spread of infectious agents.

- Contents:
- 1 reusable mask
 - disposable refill filters.
- Filter features:
- filter compliant with EN 14683 technical standard classified as Type IIR;
 - BFE > 98%;
 - Replace the filter after each day of use (12h/approx.).

The mask is indicated both for professional use by health workers and for personal use. Indication for use: it limits the emission by air of microorganisms present in the saliva droplets released while speaking, coughing or sneezing.

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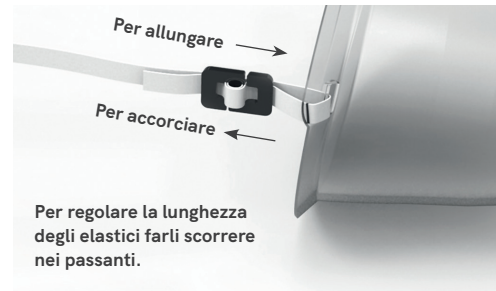
Wash the mask before using it with water and detergent or soap.

How to wear the mask:

- Wash your hands thoroughly;
- Place the mask on your face;
- Pass the elastic behind the nape and around the head or behind the ears depending on how you prefer to wear the face mask;
- In case of use of glasses, to avoid fogging, wear always first the mask making sure that it adheres correctly to the nasal septum, then wear the glasses by placing them on the mask.

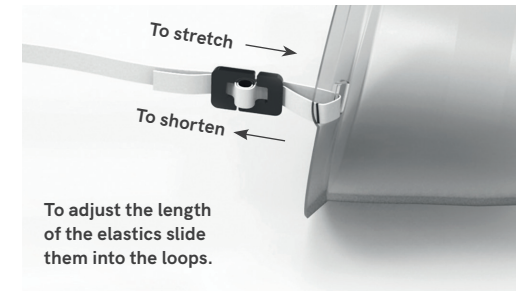
How to remove and store the mask:

- Remove the mask carefully by touching only the edges and elastics;
- Avoid touching the front of the filter as it could be contaminated;
- Discard the filter after use it, if possible in the appropriate collector for the disposal of sanitary material and in any case according to the indications of your municipality. Do not disperse in the environment after use;
- Clean and sanitize the mask before storing it; wash and sanitize thoroughly with soap and water or antibacterial detergent;
- Wash your hands thoroughly.

How to apply/replace the filter:

- Wash and sanitize your hands thoroughly;
- Remove the filter from its packaging;
- Remove the rings from the face mask;
- Place the filter on the external ring of the mask;
- Place the inner tightening ring on the filter and turn external ring clockwise (bayonet lock);
- Insert in the mask the two rings thus joined together.

How to adjust the elastics:

Warnings and/or precautions to be taken:

do not use if you experience discomfort or allergy symptoms on the skin. Carefully follow the instructions for use for correct use. Incorrect use can compromise the functionality of the device. Use this product only for the indicated use, any other use is to be considered improper. Do not use the product if damaged. The device must not come into contact with oils, fats and aromatic/aliphatic substances such as solvents, fuels, etc. Do not disperse in the environment after use. The device does not require special disposal procedures; it is necessary to refer to local regulations for the disposal of unsorted waste. Without fiberglass, it does not contain phthalates, minimal resistance to respiratory flow, high filtration capacity, adaptable to the face, breathable.

The kit includes:

- Internal / external clamping ring;
- Mask;
- Elastic and loops;
- Easily replaceable disposable refill filters.

Keep these instructions carefully, essential for the correct and safe use of the medical device.